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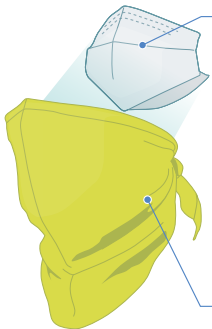
Can be worn continuously when spending long periods of time outdoors.

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- ・ Blocks and breaks up allergens, such as pollen and mites. • Superior breathability even when worn for long periods of time. • Articulated to fit comfortably on the face. • Deodorizing effect helps reduce odors even when worn for long periods of time. • Reusable after washing. • Hand wash filter using a mild soap. • Do NOT reuse filter if using to prevent colds and viruses.

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Utilizes a quick drying material with superior breathability.

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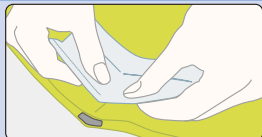
- Uses Wickron[®] COOL for super breathability even when worn for long periods of time. • Optimal for wicking moisture and quick-drying. • Blocks UV rays making it great for outdoor use.

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Replacement Filter Mask (#1124403) available when replacing filter.

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① Peel off the filter's release paper and stick it to the black spot on the inside of the mask.

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② Cover your nose and tie it at the back of your head.

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③ Adjust the wire at the top of the mask to align the shape of your nose.

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・ Do NOT use this product in areas where harmful particles or gases are found. • If itching, rash, or other symptoms develop, stop using and consult a physician. • Do NOT use near fire. • During use, be careful of wire located at top of field mask. • Keep out of the reach of children. • For the best results, eliminate gaps between face and mask. • Due to manufacturing process, filter color may vary. Color variation has no effect on product performance. • This product does NOT completely eliminate the risk of illnesses. To further reduce the risk of illness, additional measures are recommended, such as hand washing and gargling.

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フィルター: MADE IN JAPAN
表生地: 本体に記載
Filter: MADE IN JAPAN
Outer fabric: Description written on the product

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